

SafeSleep™ Cleaning Recommendations

- **ROUTINE CARE:**

Wipe clean with neutral soap suds and lukewarm water. Rinse and dry.

- **HARD TO CLEAN AREAS:**

Soft sponge with liquid cleaner as specified on manufacturer's product label.
DO NOT USE HARSH CLEANERS OR SOLVENTS.

LAUNDERING IS NOT RECOMMENDED

- **DISINFECTION:**

Most phenolic or quaternary type disinfectants can be used on SafeSleep™ fabric. User must dilute disinfectants and germicides in accordance with manufacturer's instructions.

- **BLOOD CONTAMINATION:**

Contamination with blood can be disinfected with a 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC (US Dept. of Health and Human Services, February 1989).

- **ODOR CONTROL:**

To address odors, follow cleaning recommendations. Routine care may help with odors over time. If odors persist, it may be time for a new mattress.

- **WARNING:**

Iodophor type disinfectants (Betadine, for example) will strain SafeSleep™ fabric.

- **WARNING:**

Prolonged contact with bleach solutions, improper dilution ratios for any cleaning agent, or poor rinsing of fabric surface after disinfecting and cleaning may cause damage to both the appearance and performance of this fabric.